KirPACS undervisning
Oktober 2011
jan@scanread.eu

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3. skriv eventuelt supplerende information

4. tryk på Rekvirer rapport

The image shows a screenshot of a web browser window titled "Job detaljer - Mozilla Firefox". The address bar shows the URL "http://www.kirpacs.dk/pacs/JobServlet". The page content includes a section for "Hoved arbejdsdiagnose" with checkboxes for "Congenit", "Trauma", and "Inflammation". Below this is a "Rapport" section with a "Supplerende information" field containing "Demo". A link "Report455.pdf" is visible at the bottom of the page. A blue callout bubble with the text "Klik på rapporten" points to this link.

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The screenshot shows the KirPACS web application interface. The main window is titled "Studier - Mozilla Firefox" and displays the URL "https://www.kirpacs.dk/pacs/Start". The interface includes a navigation menu on the left with options like "Søgning", "Arkiv", "Egen klinisk", "Filter", and "Alle studier". The main content area shows the "User Profile" for "Rygcenter Herning - Kiropraktisk Klinik - Klaus Doktor" with fields for "Organisation", "User ID" (dummy), "Password", "Substitute", and "Create user ID for colleague". A "Bruger" menu is visible on the right with options: "Log af", "Ændre profil", "Ændre stationer", and "Hjælp".

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Søgefelter


Sorteringsfelter

The screenshot shows the 'Explorer' window of the ClearCanvas Workstation 2.0 SP1. The interface is divided into several sections:

- Search Section:** Contains search filters for Patient ID, Name, Accession#, Study Description, Study Date (From/To), Referring Physician, and Modality. It includes buttons for 'Search', 'Today', 'Clear', and 'Last 7 days'.
- Servers Section:** A tree view on the left showing 'My Studies', 'My Servers', and 'DPACS'.
- Study List Section:** A table titled '25 studies found' with columns for Patient ID, Last Name, First Name, DOB, Accession Number, Study Date, Description, Modal, Referring Physician, and Instance. A blue wavy pattern is overlaid on the first few rows of the table.
- Bottom Section:** An 'Import Activity' button.

Patient ID	Last Name	First Name	DOB	Accession Number	Study Date	Description	Modal	Referring Physician	Instance
1802781650	BLIDDAL	PETER FRIED			23-mar-2...	Col thoracalis	CR		2
2...					24-mar-2...	Col lumbalis	CR		3
					23-mar-2...	Col lumbalis	CR		2
					22-mar-2...	Col thoracalis	CR		2
					25-mar-2...	Col lumbalis, B...	CR		3
					25-mar-2...	Overekstremitet	CR		2
					25-mar-2...	Col lumbalis	CR		2
					22-mar-2...	B...	CR		1
					23-mar-2...	Col lumbalis	CR		2
					24-mar-2...	Col lumbalis	CR		3
					18-mar-2...	Col lumbalis, B...	CR		3
					21-mar-2...	Col lumbalis, B...	CR		6
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3
					22-mar-2...	Col thoracalis, Col cer...	CR		5
					22-mar-2...	B..., Col lumbalis	CR		4
					21-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3

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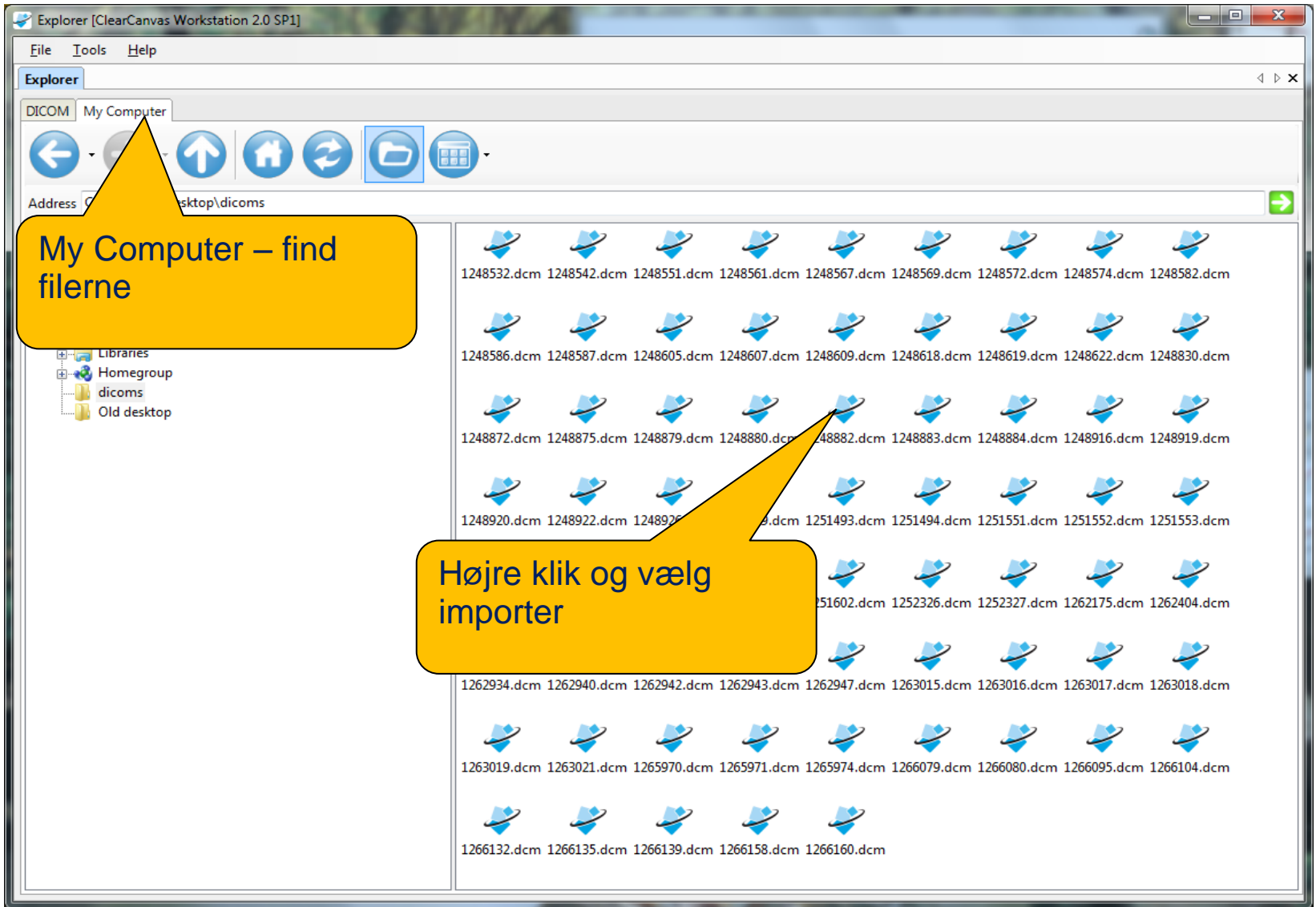The screenshot shows the ClearCanvas Workstation 2.0 SP1 Explorer interface. The main window is titled 'Explorer [ClearCanvas Workstation 2.0 SP1]' and has a menu bar with 'File', 'Tools', and 'Help'. The 'Explorer' tab is active, showing a search interface with fields for Patient ID, Name, Accession#, Study Description, Study Date (From/To), Referring Physician, and Modality. There are buttons for 'Search', 'Today', 'Clear', and 'Last 7 days'. Below the search interface, there are two main sections: 'Servers' on the left and '25 studies found on My Studies' on the right. The 'Servers' section shows a tree view with 'My Studies', 'My Servers', and 'DPACS'. The '25 studies found on My Studies' section displays a table of study results. A 'Send Study' dialog box is open in the foreground, showing a tree view with 'My Servers' and 'DPACS' selected. The table below contains the following data:

Patient ID	Last Name	First Name	DOB	Accession Number	Study Date	Description	Modal	Referring Physician	Instance
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3
					25-mar-2...	Col lumbalis,Bjokken	CR		3
					23-mar-2...	Col cervicalis	CR		3
					21-mar-2...	Col lumbalis,Bjokken,...	CR		6
					21-mar-2...	Col lumbalis	CR		2
					24-mar-2...	Col lumbalis	CR		3
					22-mar-2...	Bjokken	CR		1
					24-mar-2...	Col lumbalis	CR		3
					22-mar-2...	Bjokken,Col lumbalis	CR		4
					22-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col thoracalis	CR		2
					25-mar-2...	Col lumbalis	CR		2
					21-mar-2...	Col cervicalis	CR		3
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col lumbalis	CR		2

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