# KirPACS undervisning Marts 2011

jan@scanread.eu

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The screenshot shows the KirPACS web application interface. At the top, there is a navigation bar with the logo 'N KKB KirPACS', the time '16:24:19', and the user ID 'User ID: dummy'. Below this is a search section titled 'Søgning' with input fields for 'Jobid', 'Patientnavn' (containing 'aa'), and 'Fra', along with a 'Søg' button. There are also checkboxes for 'Hele KirPACS' and 'Ryd felter', and an 'Opdater' button. Below the search section is a table titled 'Studies' with columns for 'Job ID', 'Optagelse dato', 'Patientid', 'Patientnavn', 'Beskrivelse', and 'Modalit'. A blue wavy pattern obscures the 'Patientid' column, with the word 'anonymiseret' written over it. The table contains several rows of study data, including 'Col lumbalis' and 'Col lumbalis, Bækken, Col cervicalis'. At the bottom of the interface, there is a status bar with 'Done' and 'Apache-Coyote/1.1'.

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Done Apache-Coyote/1.1

# Gennemgang af KirPACS - rapport rekvirering i standardiseret form

The screenshot shows a web browser window with the URL <http://www.kirpacs.dk/pacs/JobServlet>. The page title is "Job detaljer". The main content area is titled "Rapport rekvirering" and contains several sections for selecting report options:

- Region og projektioner**: A section with multiple columns of checkboxes for different anatomical regions and projections, such as "Columna cervicalis", "Columna lumbalis", "Columna thoracalis", "Articulatio humeri (skulderled)", "Articulatio coxae (hoftelæd)", "Articulatio genu (knæled)", "Articulatio Radiocarpea (håndled)", "Manus (hånd)", and "Hoved arbejdsdiagnose".
- Supplerende information**: A text input field for additional information.
- Sammenlign med Rapport leverander**: A dropdown menu currently set to "ingen".
- Buttons**: Two buttons labeled "Rekvirer eksternt rapport" and "Rekvirer eksternt rapport - via rekvirering".

Four dark blue callout boxes with white text provide instructions:

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At the bottom of the page, there is a link: "Link til ClearCanvas download [ClearCanvas Workstation 2.0 SP1](#)". The browser status bar shows "Done" and "Apache-Coyote/1.1".

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The image shows a web browser window titled "Job detaljer - Mozilla Firefox" displaying a medical report page. The page has a purple header with "Job detaljer" and "Hoved arbejdsdiagnose". Below the header, there are checkboxes for "Congenit", "Trauma", and "Inflammation". A "Rapport" section is visible, along with a "Supplerende information" field containing "Demo". A blue callout bubble with the text "Klik på rapporten" points to a link labeled "Report455.pdf" in the "Rapport" section. At the bottom of the browser window, there is a link to "ClearCanvas download ClearCanvas Workstation 2.0 SP1".

An Adobe Reader window titled "455\_Report455-1.pdf - Adobe Reader" is open, showing the PDF report. The report header includes the KirPACS logo and "N/KKB". The patient information is redacted with a blue wavy pattern. The report content includes:

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The screenshot shows two browser windows. The main window is 'Alle jobs - Mozilla Firefox' at 'http://www.kirpacs.dk/pacs/Start'. An inset window is 'User Details - Mozilla Firefox' at 'http://www.kirpacs.dk/pacs/ProfileServlet'. The 'User Profile' form shows:

- Organisation: Rygcenter Herning - Kiropraktisk Klinik - Klaus Doktor
- User ID: dummy
- Password: [masked]
- Password: [masked]
- Substitute: none
- Buttons: Update, Create userid

The main window shows a table of reports with columns for 'Rapport klar' and 'Rapport Note'. A callout bubble points to the 'Log af' button in the top right. Another callout bubble points to the 'Update' button in the 'User Profile' window.

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Søgefelter


Sorteringsfelter

The screenshot shows the ClearCanvas Workstation 2.0 SP1 interface. At the top, there is a search bar with the title "Search". Below it, there are several input fields for "Patient ID", "Name", "Accession#", and "Study Description". There are also date pickers for "Study Date (From)" and "Study Date (To)", and a field for "Referring Physician". On the right side of the search bar, there are buttons for "Search", "Today", "Clear", and "Last 7 days".

Below the search bar, there is a "Servers" panel on the left showing a tree view with "My Studies", "My Servers", and "DPACS". The main area displays a list of 25 studies found. The list has columns for "Patient ID", "Last Name", "First Name", "DOB", "Accession Number", "Study Date", "Description", "Modal", "Referring Physician", and "Instance". A blue wavy pattern is overlaid on the first few rows of the table.

Patient ID	Last Name	First Name	DOB	Accession Number	Study Date	Description	Modal	Referring Physician	Instance
1802781650	BLIDDAL	PETER FRIED			23-mar-2...	Col thoracalis	CR		2
25					24-mar-2...	Col lumbalis	CR		3
					23-mar-2...	Col lumbalis	CR		2
					22-mar-2...	Col thoracalis	CR		2
					25-mar-2...	Col lumbalis, B3kken	CR		3
					25-mar-2...	Overekstremitet	CR		2
					25-mar-2...	Col lumbalis	CR		2
					22-mar-2...	B3kken	CR		1
					23-mar-2...	Col lumbalis	CR		2
					24-mar-2...	Col lumbalis	CR		3
					18-mar-2...	Col lumbalis, B3kken	CR		3
					21-mar-2...	Col lumbalis, B3kken, ...	CR		6
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3
					22-mar-2...	Col thoracalis, Col cer...	CR		5
					22-mar-2...	B3kken, Col lumbalis	CR		4
					21-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3

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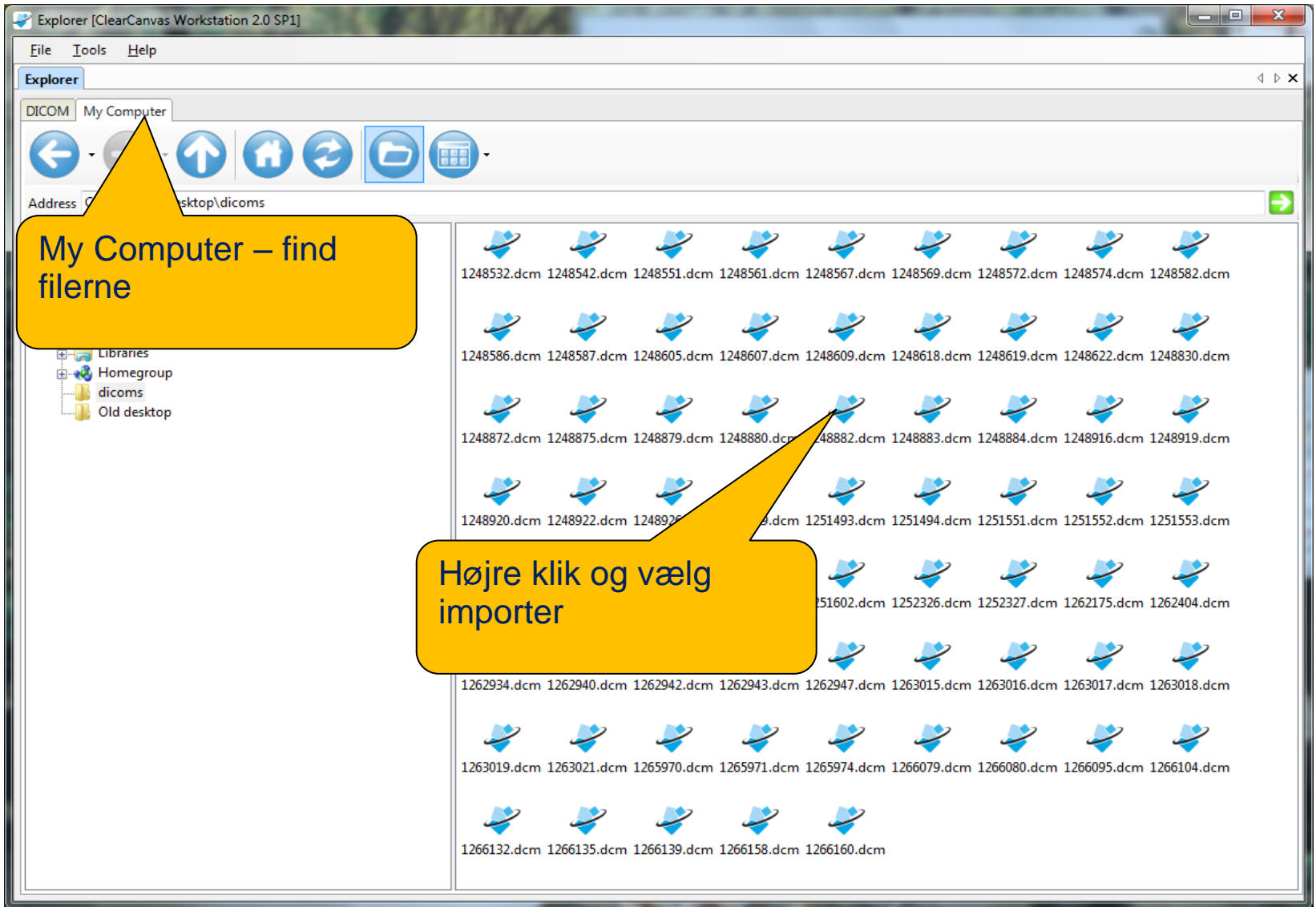The screenshot shows the ClearCanvas Workstation 2.0 SP1 Explorer interface. The main window is titled 'Explorer [ClearCanvas Workstation 2.0 SP1]' and has a menu bar with 'File', 'Tools', and 'Help'. The 'Explorer' tab is active, showing a search interface for DICOM studies. The search criteria include Patient ID, Name, Accession#, Study Description, Study Date (From/To), Referring Physician, and Modality. A 'Send Study' dialog box is open, showing a tree view of 'My Servers' with 'DPACS' selected. The main window displays a table of 25 studies found on 'My Studies'.

Patient ID	Last Name	First Name	DOB	Accession Number	Study Date	Description	Modal	Referring Physician	Instance
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col cervicalis	CR		3
					25-mar-2...	Col lumbalis,Bjokken	CR		3
					23-mar-2...	Col cervicalis	CR		3
					21-mar-2...	Col lumbalis,Bjokken,...	CR		6
					21-mar-2...	Col lumbalis	CR		2
					24-mar-2...	Col lumbalis	CR		3
					22-mar-2...	Bjokken	CR		1
					24-mar-2...	Col lumbalis	CR		3
					22-mar-2...	Bjokken,Col lumbalis	CR		4
					22-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col thoracalis	CR		2
					25-mar-2...	Col lumbalis	CR		2
					21-mar-2...	Col cervicalis	CR		3
					24-mar-2...	Col lumbalis	CR		2
					23-mar-2...	Col lumbalis	CR		2

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